

Service Report



WORK ORDER ID: CWKD6982953

WORK ORDER TYPE: Preventative Maintenance

CUSTOMER: UNIVERSITY OF KS MED CTR-CTSU | **DEPARTMENT:** 146148-UNIVERSITY OF KS MED CTR-CTSU-BIOMED | **ACCOUNT NUMBER:** 146148

CONTACT:

CUSTOMER PO#:

CONTACT PHONE:

CUSTOMER REFERENCE:

BILLING ADDRESS: PO Box 3160 Mail Stop 6011 Kansas City, KS 66103

SHIPPING ADDRESS: 4350 Shawnee Mission Parkway Mail Stop 6011 Fairway, KS 66205

AGILITY ASSET ID : 1887906 | **SERIAL #:** 100093460821 | **VENDOR SITE ID:** | **CUSTOMER ASSET ID:**

ASSET TYPE: MONITORS, PHYSIOLOGIC, VITAL SIGNS | **MANUFACTURER:** WELCH ALLYN INC | **MODEL:** CONNEX SPOT MONITOR

SERVICE REQUESTED: PM DUE

SERVICE PROVIDED TYPE: Scheduled Maintenance

WORK COMPLETED: 01-22-2025

LABOR

| SERVICE PROVIDER | CATEGORY | HOURS | PERIOD DEFINITION | SERVICE DATE |
|------------------|----------------|-------|-------------------|--------------|
| Stephanie Dye | PM Not Covered | 0.5 | M-F 8-5 | 01/22/25 |

SERVICE PROVIDED DETAILS

UNIT PASSED ALL REQUIRED TESTING AT THIS TIME

TEST EQUIPMENT

| PREFIX | UNIT | DESCRIPTION | CALIBRATION DATE |
|--------|------|--|------------------|
| SP3 | 0105 | VITAL SIGNS PATIENT SIMULATOR, PROSIM 3 | |
| SP4 | 0105 | SPO2 SIMULATOR, FLUKE SPOTLIGHT,PROSIM 4 | |

CHECKLIST**PROCEDURE: MONITORS, BEDSIDE, PHYSIOLOGIC****QUESTION****ANSWER**

| | |
|--|------|
| 01. Check AC plug/line cord/strain relief | Pass |
| 02. Measure chassis ground resistance (milliOhms) | 1 |
| 03. Measure chassis leakage current (microAmps) | 68 |
| 04. Patient Leakage (microAmps) | N/A |
| 05. Check outercase, line cords, rolling stands, wall mounts, modular accessories and interconnecting cables | Pass |
| 06. Check patient interface connections (ECG, IBP, SPO2, Temp, CO2 and NIBP) | Pass |
| 07. Check display panel, feet, screws and handle | Pass |
| 08. Keypad/Control Knob Test | Pass |
| 09. Recorder Test | Pass |
| 10. Display Tests | Pass |
| 11. Pixel Test | Pass |
| 12. Color Test | Pass |
| 13. NIBP Tests - Overpressure Voltage Test | Pass |
| 14. NIBP Tests - Static Pressure Calibration | Pass |
| 15. NIBP Tests - Motor Pump Test | Pass |
| 16. NIBP Tests - Leak Test | Pass |
| 17. NIBP Tests - Dump Test (Adult, Neonate) | Pass |
| 18. NIBP Tests - Bleed Rate Test | Pass |
| 19. NIBP Tests - Overpressure Test (Adult, Pediatric, Neonate) | Pass |
| 20. Voltage and Battery Operation Verification | Pass |
| 21. ECG Tests - Initialization | Pass |
| 22. ECG Tests - Leads Off | Pass |
| 23. ECG Tests - Pacer Detect | Pass |
| 24. ECG Tests - Heart Rate | Pass |
| 25. ECG Tests - Alarms | Pass |
| 26. IBP 1 and IBP 2 Verification | Pass |
| 27. Temperature Verification | Pass |
| 28. SPO2 Verification | Pass |
| 29. SPO2 Alarm | Pass |
| 30. NIBP Verification | Pass |
| 31. CO2 Operation Verification | Pass |

Customer Signature:

Date:

Stephanie Dye

1/22/25

Technician Signature:

Date: