Service Report



WORK ORDER ID: CWKD6982164

WORK ORDER TYPE: Preventative Maintenance

CUSTOMER: UNIVERSITY OF KS MED CTR-CTSU | DEPARTMENT: 146148-UNIVERSITY OF KS MED CTR-CTSU-BIOMED | ACCOUNT NUMBER: 146148

CONTACT: CUSTOMER PO#:

CONTACT PHONE: CUSTOMER REFERENCE:

BILLING ADDRESS: PO Box 3160 Mail Stop 6011 Kansas City, KS 66103 SHIPPING ADDRESS: 4350 Shawnee Mission Parkway Mail Stop 6011 Fairway, KS 66205

AGILITI ASSET ID: 1887900 | SERIAL #: 100093500821 | VENDOR SITE ID: | CUSTOMER ASSET ID:

ASSET TYPE: MONITORS, PHYSIOLOGIC, VITAL SIGNS | MANUFACTURER: WELCH ALLYN INC | MODEL: CONNEX SPOT MONITOR

SERVICE REQUESTED: PM DUE

SERVICE PROVIDED TYPE: Scheduled Maintenance

WORK COMPLETED: 01-22-2025

LABOR

SERVICE PROVIDER	CATEGORY	HOURS	PERIOD DEFINITION	SERVICE DATE
Stephanie Dye	PM Not Covered	0.6	M-F 8-5	01/22/25

SERVICE PROVIDED DETAILS

UNIT PASSED ALL REQUIRED TESTING AT THIS TIME

TEST EQUIPMENT PREFIX UNIT DESCRIPTION SP3 0105 VITAL SIGNS PATIENT SIMULATOR, PROSIM 3 SP4 0105 SP02 SIMULATOR, FLUKE SPOTLIGHT, PROSIM 4

CHECKLIST

PROCEDURE: MONITORS, BEDSIDE, PHYSIOLOGIC QUESTION **ANSWER** 01. Check AC plug/line cord/strain relief **Pass** 02. Measure chassis ground resistance (milliOhms) 1 03. Measure chassis leakage current (microAmps) 72 04. Patient Leakage (microAmps) N/A 05. Check outercase, line cords, rolling stands, wall mounts, modular accessories and interconnecting cables **Pass** 06. Check patient interface connections (ECG, IBP, SPO2, Temp, CO2 and NIBP) **Pass** 07. Check display panel, feet, screws and handle **Pass** 08. Keypad/Control Knob Test **Pass** 09. Recorder Test **Pass** 10. Display Tests **Pass** 11. Pixel Test **Pass** 12. Color Test **Pass** 13. NIBP Tests - Overpressure Voltage Test **Pass** 14. NIBP Tests - Static Pressure Calibration **Pass** 15. NIBP Tests - Motor Pump Test **Pass** 16. NIBP Tests - Leak Test **Pass** 17. NIBP Tests - Dump Test (Adult, Neonate) **Pass** 18. NIBP Tests - Bleed Rate Test **Pass** 19. NIBP Tests - Overpressure Test (Adult, Pediatric, Neonate) **Pass** 20. Voltage and Battery Operation Verification **Pass** 21. ECG Tests - Initialization Pass 22. ECG Tests - Leads Off **Pass** Pass 23. ECG Tests - Pacer Detect 24. ECG Tests - Heart Rate **Pass** 25. ECG Tests - Alarms **Pass** 26. IBP 1 and IBP 2 Verification **Pass** 27. Temperature Verification **Pass** 28. SPO2 Verification **Pass** 29. SPO2 Alarm **Pass** 30. NIBP Verification **Pass** 31. CO2 Operation Verification **Pass**

Stephanie	Dye
•	0

1/22/25

Technician Signature:

Date: