

# Service Report



## WORK ORDER ID: CWKD4070816

**WORK ORDER TYPE:** Preventative Maintenance

**CUSTOMER:** UNIV OF KS MED CTR-CTSU RNBW | **DEPARTMENT:** 177820-UNIV OF KS MED CTR-CTSU RNBW-ROUNDS CTSU RNBW | **ACCOUNT NUMBER:** 177820

**CONTACT:**

**CUSTOMER PO#:**

**CONTACT PHONE:**

**CUSTOMER REFERENCE:**

**BILLING ADDRESS:** 3901 Rainbow Blvd MS 1077 2012 Suder Kansas City, KS 66160 | **SHIPPING ADDRESS:** 3901 Rainbow Blvd MS 1077 2012 Suder Kansas City, KS 66160

**AGILITY ASSET ID :** 1887871 | **SERIAL #:** 100088374021 | **VENDOR SITE ID:** | **CUSTOMER ASSET ID:**

**ASSET TYPE:** MONITORS, PHYSIOLOGIC, VITAL SIGNS | **MANUFACTURER:** WELCH ALLYN INC A SUBSIDIARY OF HILLROM HOLDINGS INC | **MODEL:** SPOT VITAL SIGNS 4400

**SERVICE REQUESTED:** Asset Onboarding - Initial Preventative Maintenance for 1887871

**SERVICE PROVIDED TYPE:** Scheduled Maintenance

**WORK COMPLETED:** 01-19-2024

### LABOR

SERVICE PROVIDER	CATEGORY	HOURS	PERIOD DEFINITION	SERVICE DATE
Stephanie Harper	PM Not Covered	0.22	M-F 8-5	01/19/24
Marcus Boatwright	PM Not Covered	0.22	M-F 8-5	01/19/24

### SERVICE PROVIDED DETAILS

Completed SM passed all inspection testing

### TEST EQUIPMENT

PREFIX	UNIT	DESCRIPTION	CALIBRATION DATE
PR3	473	OXIMETER SIMULATOR	04/14/23

**CHECKLIST**

**PROCEDURE: MONITORS, BEDSIDE, PHYSIOLOGIC**

<b>QUESTION</b>	<b>ANSWER</b>
01. Check AC plug/line cord/strain relief	Pass
02. Measure chassis ground resistance ( milliOhms )	1
03. Measure chassis leakage current ( microAmps )	88
04. Patient Leakage ( microAmps )	N/A
05. Check outercase, line cords, rolling stands, wall mounts, modular accessories and interconnecting cables	Pass
06. Check patient interface connections (ECG, IBP, SPO2, Temp, CO2 and NIBP)	Pass
07. Check display panel, feet, screws and handle	Pass
08. Keypad/Control Knob Test	Pass
09. Recorder Test	NA
10. Display Tests	Pass
11. Pixel Test	Pass
12. Color Test	Pass
13. NIBP Tests - Overpressure Voltage Test	Pass
14. NIBP Tests - Static Pressure Calibration	Pass
15. NIBP Tests - Motor Pump Test	Pass
16. NIBP Tests - Leak Test	Pass
17. NIBP Tests - Dump Test (Adult, Neonate)	Pass
18. NIBP Tests - Bleed Rate Test	Pass
19. NIBP Tests - Overpressure Test (Adult, Pediatric, Neonate)	Pass
20. Voltage and Battery Operation Verification	Pass
21. ECG Tests - Initialization	Pass
22. ECG Tests - Leads Off	Pass
23. ECG Tests - Pacer Detect	Pass
24. ECG Tests - Heart Rate	Pass
25. ECG Tests - Alarms	Pass
26. IBP 1 and IBP 2 Verification	NA
27. Temperature Verification	Pass
28. SPO2 Verification	Pass
29. SPO2 Alarm	Pass
30. NIBP Verification	Pass
31. CO2 Operation Verification	NA

---

Customer Signature:

Date:

---

Technician Signature:

Date: