

Service Report



WORK ORDER ID: CWKD4070839

WORK ORDER TYPE: Preventative Maintenance

CUSTOMER: UNIV OF KS MED CTR-CTSU RNBW | **DEPARTMENT:** 177820-UNIV OF KS MED CTR-CTSU RNBW-ROUNDS CTSU RNBW | **ACCOUNT NUMBER:** 177820

CONTACT:

CUSTOMER PO#:

CONTACT PHONE:

CUSTOMER REFERENCE:

BILLING ADDRESS: 3901 Rainbow Blvd MS 1077 2012 Suder Kansas City, KS 66160 | **SHIPPING ADDRESS:** 3901 Rainbow Blvd MS 1077 2012 Suder Kansas City, KS 66160

AGILITI ASSET ID : 1887882 | **SERIAL #:** A18044501468 | **VENDOR SITE ID:** | **CUSTOMER ASSET ID:**

ASSET TYPE: MONITORS, PHYSIOLOGIC, MULTIPURPOSE, BEDSIDE, CONFIGURED | **MANUFACTURER:** AXIA SURGICAL | **MODEL:** V1050T PATIENT MONITOR

SERVICE REQUESTED: Asset Onboarding - Initial Preventative Maintenance for 1887882

SERVICE PROVIDED TYPE: Scheduled Maintenance

WORK COMPLETED: 01-19-2024

LABOR

SERVICE PROVIDER	CATEGORY	HOURS	PERIOD DEFINITION	SERVICE DATE
Marcus Boatwright	PM Not Covered	0.22	M-F 8-5	01/19/24
Stephanie Harper	PM Not Covered	0.22	M-F 8-5	01/19/24

SERVICE PROVIDED DETAILS

Completed SM and passed all inspection testing. Also, a note on the machine says SP02 is non-functional.

TEST EQUIPMENT

PREFIX	UNIT	DESCRIPTION	CALIBRATION DATE
SB6	223	SAFETY ANALYZER, FLUKE ESA612	04/25/23
PR3	473	OXIMETER SIMULATOR	04/14/23

CHECKLIST**PROCEDURE: MONITORS, BEDSIDE, PHYSIOLOGIC****QUESTION****ANSWER**

01. Check AC plug/line cord/strain relief	Pass
02. Measure chassis ground resistance (milliOhms)	115
03. Measure chassis leakage current (microAmps)	40
04. Patient Leakage (microAmps)	N/A
05. Check outercase, line cords, rolling stands, wall mounts, modular accessories and interconnecting cables	Pass
06. Check patient interface connections (ECG, IBP, SPO2, Temp, CO2 and NIBP)	Pass
07. Check display panel, feet, screws and handle	Pass
08. Keypad/Control Knob Test	Pass
09. Recorder Test	NA
10. Display Tests	NA
11. Pixel Test	NA
12. Color Test	NA
13. NIBP Tests - Overpressure Voltage Test	Pass
14. NIBP Tests - Static Pressure Calibration	Pass
15. NIBP Tests - Motor Pump Test	Pass
16. NIBP Tests - Leak Test	Pass
17. NIBP Tests - Dump Test (Adult, Neonate)	Pass
18. NIBP Tests - Bleed Rate Test	Pass
19. NIBP Tests - Overpressure Test (Adult, Pediatric, Neonate)	Pass
20. Voltage and Battery Operation Verification	Pass
21. ECG Tests - Initialization	NA
22. ECG Tests - Leads Off	NA
23. ECG Tests - Pacer Detect	NA
24. ECG Tests - Heart Rate	NA
25. ECG Tests - Alarms	NA
26. IBP 1 and IBP 2 Verification	NA
27. Temperature Verification	NA
28. SPO2 Verification	NA
29. SPO2 Alarm	NA
30. NIBP Verification	Pass
31. CO2 Operation Verification	NA

Customer Signature:

Date:

Technician Signature:

Date: